



Virtually pain free¹ testing

Accu-Chek Lancing System

A *gentle* touch on your fingertips



Accu-Chek FastClix

Key features:

- 1-Click Action
- Drum-type lancets: avoiding accidental needle injury
- Clixmotion® technology offers virtually pain free lancing experience



Accu-Chek SoftClix

Key features:

- Simple 2-step lancing: fast lancing for convenience
- Clixmotion® technology offers virtually pain free lancing experience

Tips & tricks for a gentle lancing experience²

- For first-timers, **start by selecting small penetration depth** to find the optimal depth that obtain sufficient blood sample
- **Use a new lancet for every new test!** Used lancets are blunt and increase risk of infection
- **Gently massage user's hand** before pricking to improve blood circulation
- Choose to **prick at the side of the fingertip**



VIDEO: How to use FastClix device



VIDEO: How to use SoftClix device

Purchase Accu-Chek FastClix online at:



Purchase Accu-Chek SoftClix online at:



The Accu-Chek FastClix and Accu-Chek Softclix lancing devices are intended for personal use only. The same device must not be used on multiple patients.

References:

1. Kocher, S, et al., Comparison of Lancing Devices for Self-Monitoring of Blood Glucose Regarding Lancing Pain_J Diabetes Sci Technol _2009; Sep. 3(5):1136-1143
2. Accu-Chek FastClix and Accu-Chek Softclix package insert

ACCU-CHEK, SOFTCLIX and FASTCLIX are trademarks of Roche. © 2020 Roche Diabetes Care.

www.accu-chek.com.sg

Roche Diabetes Care Asia Pacific Pte. Ltd.
8 Kallang Avenue #11-07/09 Aperia Tower 1 Singapore 339509

ACCU-CHEK®