



# Diabetes data and detailed reports at your fingertips to share with your healthcare professional

## Getting started with the mySugr app

- 1 Download the mySugr app.  
- 2 Create your mySugr account.
- 3 Unlock mySugr Pro for free to access the advanced PDF report feature and more:



Pair your Accu-Chek® Guide or Instant meter for unlimited access to Pro.


Go to “Connections” on the menu to start the [pairing process](#).

- 4 Import your BG values automatically from your Accu-Chek® meter.


## Stay connected: share mySugr reports with your healthcare professional

Share PDF reports detailing 2 weeks or 3 months of your diabetes data. ([Watch tutorial here.](#))


### Producing a report and sharing the data

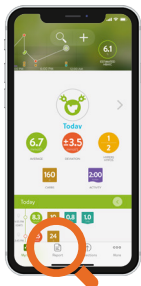


Select “Report” from the menu

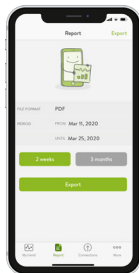

Choose the desired time period and click “Export”

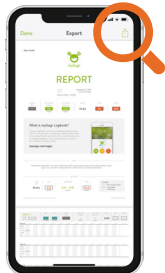

Click the share icon

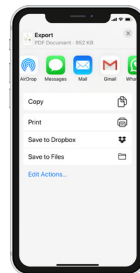

Select the app you wish to use to share the report



More than 2 million registered users.  
Average 4.6 star user experience rating.

For local customer support, contact Accu-Chek ExtraCare at 6272 9200.

ACCUCHEK, ACCUCHEK GUIDE, ACCUCHEK INSTANT and MYSUGR are trademarks of Roche.  
©2020 Roche Diabetes Care, Inc. All rights reserved.